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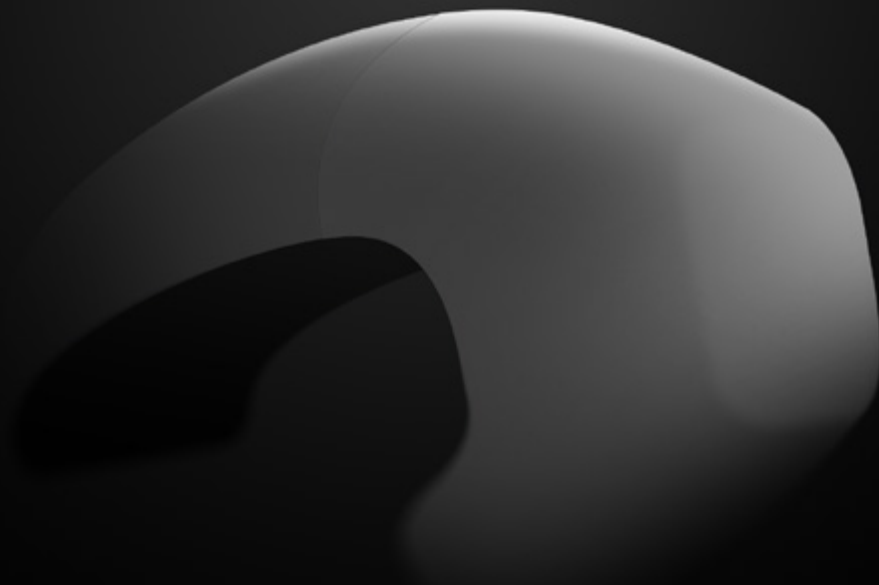
1 rok

Výrobce:

Guangzhou CHUANG ZAO MEI Technology Co., Ltd
First Branch Dehe international A501, No.2, Qixing-
gang Industrial Road, JUNHE street, Baiyun District,
Guangzhou, China
Tel.: +86 020 86016362

Content

1.	Device description.....	14
2.	Technical specifications.....	14
3.	Device appearance.....	15
4.	Device connection.....	17
5.	Operating mode.....	19
6.	Warranty.....	19

ULTRA DEZ

Application instructions

1

Device description

ULTRA DEZ device description: The invention belongs to devices for comprehensive disinfection of work surfaces in order to prevent the transmission of bacterial or fungal infection during procedures involving contact of equipment with human tissues. Decontamination rate is 99%.

The disinfectant surface is resistant to disinfection treatment. The decontaminator body is made of mechanically durable polymer. The irradiator is resistant to the impact of climatic factors during its operation. The decontaminator operates on a voltage of 230V from the mains power. Connection to the network is provided by a power cord.

2

Technical specifications

Wavelength: 270-280 nm / 390-400 nm

Power supply parameters: 5V, 3A

USB input type: Type-C

Component parts:

- USB Type C wire (2 pc.)
- tester
- power adapter
- decontaminator

3

Device appearance

Figure 1.

Figure 2.

Figure 3.

Figure 4.

3

Device appearance

Figure 5. Type-C wire


Figure 6. Power Supply Unit

Figure 7. Box

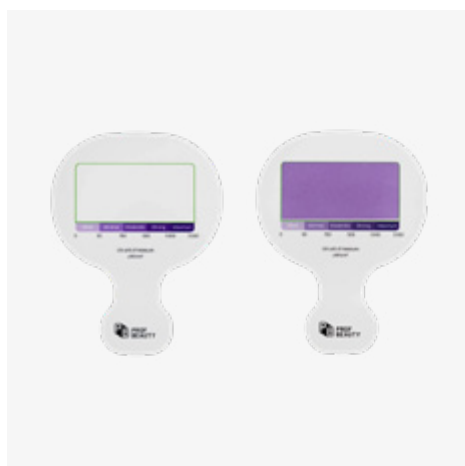Figure 8. Tester

4

Device connection

Figure 9.

Step 1.
Put on the device on
the handpiece

Figure 10.

Step 2.
Snap the device onto the back
of the handpiece

4

Device connection

Figure 11.

Step 3.
Connect the Type-C
wire to the connector

Figure 12.

Power button.
After pressing the button,
the indicator will turn white.
This means that power is supplied

5

Operating mode

1 mode. Optimal

Continuous light, cooling, blinking, cooling, steady light (LED color is blue).

2 mode. Intensive

Continuous light (LED color is yellow)

3 mode. Ultra

Continuous light (LED color is red)

6

Warranty

Device warranty:

1 year

Manufacturer:

Guangzhou CHUANG ZAO MEI Technology Co., Ltd
First Branch Dehe international A501, No.2, Qixing-gang Industrial Road, JUNHE street, Baiyun District, Guangzhou, China
Tel: +86 020 86016362

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